

Promisemed

VeriEcto® Automatic Biopsy Needles

Get Ideal
Samples



FEATURES AND BENEFITS

Promisemed VeriEcto® Automatic Biopsy Needles are designed for getting ideal samples from a wide range of soft tissue for diagnosis and causing less trauma to patients. Automatic Mode (One-step Mode) , Semi-automatic Mode (Two-step Mode) and Zero-throw Mode meet multiple demands .



Meet Multiple Demands

• Three Firing Modes

1. Automatic Mode (One-step Mode)

The cannula and trocar can be fired simultaneously, and the sample will be obtained in a short time.



01

Pull the ring twice to energize the device then penetrate the needle into the patient until it arrives at the board of the target area.



02

Press A button to get the sample.

2. Semi-automatic Mode (Two-step Mode)

The trocar will be fired firstly to allow tissue to settle in notch, thus doctors can see its position and replace the needle if necessary.



01

Pull the ring twice to energize the device then penetrate the needle into the patient until it arrives at the board of the target area.



02

Press S button to eject the trocar.




03

Press A button to eject the cutting cannula.

3. Zero-throw Mode for precise sampling

The trocar won't advance when fired which reduces the damage to deeper tissue.

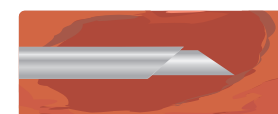

01

Penetrate the needle into the target area.



02

Pull the ring once to expose the sample notch.



03

Press A button to eject the cutting cannula.



• Two Penetration Depths



- 15mm penetration depth, 13mm sample notch



- 25mm penetration depth, 19mm sample notch

• Applicable for most organs such as breast, kidney, lung, liver, lymph gland and prostate




• Optional VeriEcto® Co-axial Biopsy Devices enhance efficiency and accuracy



Simple to use

1. Coaxial Needle Detachment Button

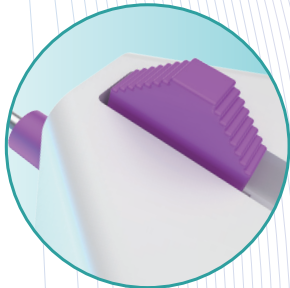

To fix the coaxial needle and reduces gas entry. Stable detachment without shaking.

2. Visual Window Design



Indicates if the needle has been fully energized.


3. Easy To Control



Pull the ring to energize the device. 50g weight for easy hold.


Get Ideal Samples

1. Dead Space

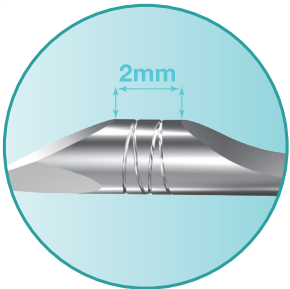

Only 2mm dead space, shorten the distance between the sampling notch and the needle tip.

2. Special Designed Tip

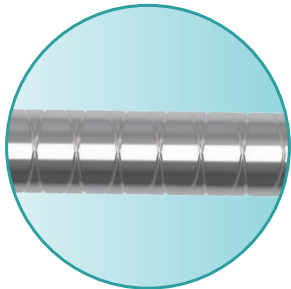

Enhances visulization under ultrasound.

3. Three-edge Needle Type



A sharp needle tip to facilitate penetration.

ORDERING INFORMATION

REF	Promisemed VeriEcto® Automatic Biopsy Needles	Promisemed VeriEcto® Co-axial Biopsy Devices
ABN2-1410	2.1 (14G) ×100mm	/
ABN2-1410C	2.1 (14G) ×100mm	2.4 (13G) ×70mm
ABN2-1413	2.1 (14G) ×130mm	/
ABN2-1413C	2.1 (14G) ×130mm	2.4 (13G) ×100mm
ABN2-1416	2.1 (14G) ×160mm	/
ABN2-1416C	2.1 (14G) ×160mm	2.4 (13G) ×130mm
ABN2-1610	1.6 (16G) ×100mm	/
ABN2-1610C	1.6 (16G) ×100mm	1.8 (15G) ×70mm
ABN2-1616	1.6 (16G) ×160mm	/
ABN2-1616C	1.6 (16G) ×160mm	1.8 (15G) ×130mm
ABN2-1620	1.6 (16G) ×200mm	/
ABN2-1620C	1.6 (16G) ×200mm	1.8 (15G) ×170mm
ABN2-1810	1.2 (18G) ×100mm	/
ABN2-1810C	1.2 (18G) ×100mm	1.4 (17G) ×70mm
ABN2-1816	1.2 (18G) ×160mm	/
ABN2-1816C	1.2 (18G) ×160mm	1.4 (17G) ×130mm
ABN2-1820	1.2 (18G) ×200mm	/
ABN2-1820C	1.2 (18G) ×200mm	1.4 (17G) ×170mm
ABN2-1825	1.2 (18G) ×250mm	/
ABN2-1825C	1.2 (18G) ×250mm	1.4 (17G) ×220mm

- ※ Apply one reference number (REF) for both the automatic biopsy needle and co-axial biopsy device to simplify ordering and ensure that the co-axial biopsy device is compatible with the automatic biopsy needle.
- ※ If you have further questions or need other specifications, please contact us.

PROMISEMED HANGZHOU MEDITECH CO., LTD.

No. 1388 Cangxing Street, Cangqian Community,
Yuhang District, Hangzhou City, 311121 Zhejiang, P.R.China
info@promisemed.ca
www.promisemed-global.com



6666-ABN2-VeriEcto-H-00 (EN)